# SteriTec Steam Chart 121/134

REF CI 108/121-134

## Directions For Use


### INDICATIONS FOR USE

The SteriTec Steam Chart 121/134 Chemical Integrator strips are designed for use in steam sterilizers operating at 121°C-134°C. When used as directed, the SteriTec Steam Chart Integrator strips give visual confirmation that sterilization conditions were met at the location in the sterilizer chamber.

### COLOR CHANGE

During steam sterilization, the four indicators spots change from purple to green.

STATED VALUES/CRITICAL PARAMETERS (As determined in a steam sterilization resistometer)		
Spot	121° C	134° C
1	8 min.	3.25 min.
2	11 min.	4.5 min.
3	17 min.	7 min.
4	30 min.	11 min.

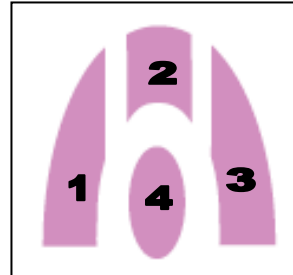### INSTRUCTIONS FOR USE

1. Place a SteriTec Steam Chart strip in each pack, peel-pouch or tray for sterilization. Place strip at a location in the package considered most difficult for steam to reach. For larger packs, place strips at several locations in pack.
2. Sterilize package as usual in a steam sterilizer.
3. Following the sterilization cycle, verify that indicator spots 1, 2, and 3 have turned from purple to green depending on the cycle length. Spot 4 will change to green in sterilization cycles exceeding 30 minutes at 121° C or 11 minutes at 134° C.
4. If spot 2 is not green, it suggests inadequate sterilization conditions and item(s) must be re-sterilized with new indicator strip(s).

### SAFETY PRECAUTIONS

- If there is any doubt about sterility of item, it must be considered NOT sterile.
- Do not use any strips whose indicator marks are not purple in color prior to exposure to steam sterilization processing.
- Not recommended for use in a "flash" sterilization cycle.

### STORAGE

SteriTec Steam Charts steam sterilization integrator strips should be stored between 50°-100° F (10°-38°C) in the re-sealable bag provided. No special storage conditions are necessary after exposure to steam sterilization conditions. Keep unexposed indicator strips in the sealed bag provided.

### EXPIRY DATE

The expiry date is printed on the product packaging.

### LOT NUMBER

A unique identification code, **LOT**, is printed on each indicator strip and packaging labeling.

### INTERFERING SUBSTANCES OR CONDITIONS

There are NO KNOWN INTERFERING SUBSTANCES OR CONDITIONS that could affect the intended use of the indicator or adversely affect the indicator performance.

### RELEASE OF TOXIC SUBSTANCES

The indicator releases NO KNOWN TOXIC SUBSTANCES in sufficient quantities to cause either a health hazard, or a hazard to the intended properties of the product being sterilized before, during or after the sterilization process.

### DECLARATION OF CONFORMITY

SteriTec Products Mfg. Co., Inc. declares that the Steam Charts Integrator conforms to all applicable portions ISO 11140-1 Class 5 integrators.

SteriTec Products Mfg. Co., Inc.  
74 Inverness Drive East  
Englewood, CO 80112 USA  
(303) 660-4201 FAX (303) 660-4213

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SteriTec Products Mfg. Co., Inc. - 74 Inverness Drive East - Englewood, CO 80112 USA (303) 660-4201 FAX (303) 660-4213